


Getting Started

1. Select a project

Click  and select project from list.

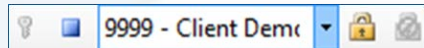


2. Open the subject list

Click on the subject list button.



3. Open images from the subject list

Click  to expand items in list.


Right click on item of interest and select "View".




Image Navigation

Switch between open visits




Click  next to the visit label & select images to display.


No images displayed

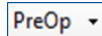

Some images displayed

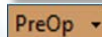

All images displayed

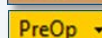

Scroll through slices

Use any of the 4 methods:

Wheel up & down 

Previous/Next Slice 

Play 

Scrollbar




Stabilization Feature (available for select projects)

Select object to stabilize

Click an object button (e.g. "C5") to stabilize.

Click "None" to remove stabilization.

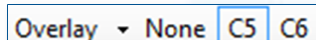

Swap displayed images

Use to view select visits in longitudinal exams

Click  next to button to select visits of interest from list.



Tip: This feature is useful for comparison with baseline images (e.g. PostOp and M24) without scrolling through all visits.


Check Local Availability of Images

Look at icons in the subject list



On local computer: Subject Visit Exam Image

Need to download

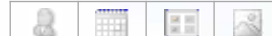

Some downloaded




All downloaded



Image Processing Tools

Select button, then use mouse controls.

 **Pan** Right click & drag

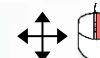

 **Zoom** Left click & drag



 **Window** Left click & drag



 **Level** Left click & drag



 **Measure**

Left click & drag to draw

Right click to delete



 **Histogram equalization**

Auto-adjusts contrast & brightness



 **Invert**

 **Show images side-by-side**

 **Multi-planar reconstruction**

Reconstructs axial, coronal, and sagittal planes from CT or MR

Tips:

- Right click on tool button to turn off. 
- Click  next to tool button for additional display options or presets.

For the Radiologist

 **Assessment panel**

 **QC message list**

Other Tools

General

 **Log in**  **Lock**

 **Log out**  **Unlock**

Subject list

 **Refresh subject list**

 **Filter subject list**

 **Select all**

 **Deselect all**

 **Download images**

 **Export images**

Window tiling

 **Tile vertically**

 **Tile horizontally**

 **Tile grid**

 **Tile cascade***

 **No tile***

 **Close all**

*Applies the next time an image is displayed.